


Intravenous (IV) Injection Training Arm – Infant

PRODUCT NO.: PC-PEE030.01

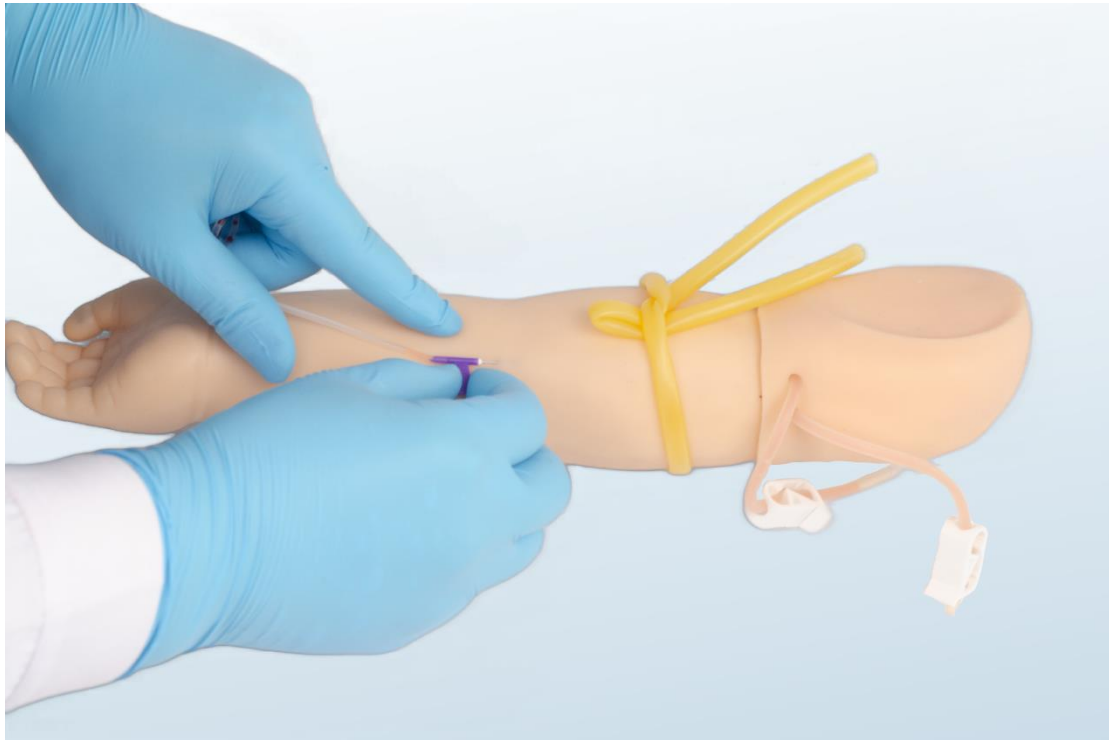

About the simulator

This IV training arm is a full-size lifelike infant arm, which realistically simulates that of a baby patient. It is specifically designed for training professionals in the practice of venipuncture.

Instructions for Use

1. Attach tubing to either vein.
2. Allow fluid to flow through arm and out the other vein.
3. Clamp off the flow of water.
4. The simulator is now ready to practice venipuncture.

Note: we recommend nothing larger than a 6 # syringe. This will extend the life of the skin and veins.



Replacing Skin and Veins

When excessive leaking occurs at the puncture sites, a new one should be installed to reduce loss of fluid.

Remove the skin:

If replacing the skin and the veins, cut the skin off. This can be done with a sharp knife or scalpel. When finished, discard the skin.

Fix the veins

Put the spare veins beside the groove of the mandrel. Leave enough space for both sides of the veins.

Fix the skin

A、 Rinse the inside of the arm skin with soap and water. Lather the arm mandrel with liquid soap.

B、 Slide the arm skin over the mandrel and work the mandrel up into the skin's fingers.

C、 Rinse off excess soap and dry.

Care and Maintenance

Clean the simulator with a damp cloth, using warm water and neutral detergent. Do not use any abrasive cleaning agents or materials.

Allow trainer to completely dry before storing.

Keep away from sunshine.

