## CPR Manikin Little Grey

### About the Simulator

The Little Grey CPR Manikin is the most realistic, economical manikin available for the training of CPR skills. Features include head tilt, sanitary airway system, realistic chest rise, adult compression resistance, palpable landmarks, and a lightweight and compact design.

### List of Components

- Base
- Simulated skin (nylon cover) with zipper
- Adult Compression Block

### Set Up Free

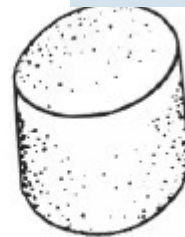
Your Little Grey Manikin is shipped disassembled. Simply follow below TWO easy steps to assemble your manikin.

1. Open the box, take out the manikin base(s) and wait until the foams recoil as normal.
2. Attaching the compression block to the middle of the base before zip the cover to make sure the proper compression resistance.


*(See figure 1 )*

### Maintenance

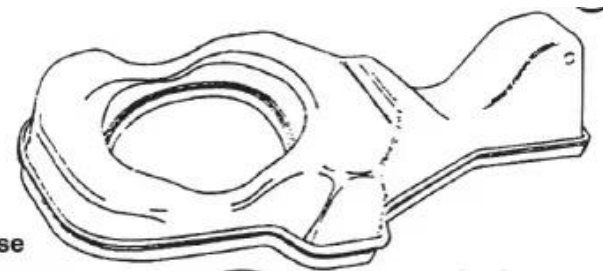No need to attach the compression block to the Base again nor install the lung bag until there is a wear & tear on the manikin. Try to gently pull the compression block out and check the recoil action to avoid contaminating the manikin by errors. In case you need further assistance, please feel free to contact us by email: [info@panckoo.com](mailto:info@panckoo.com)

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